Saljol Rollator LED-Licht

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Saljol Rollator LED light

for Saljol Carbon Rollator and Allround Rollator

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Scope of delivery
1x LED light
3x AAA battery

Product overview
1 LED light
2 Battery cover
3 AAA battery
4 Battery case
5 Snap hook
6 Magnet
7 On/off switch

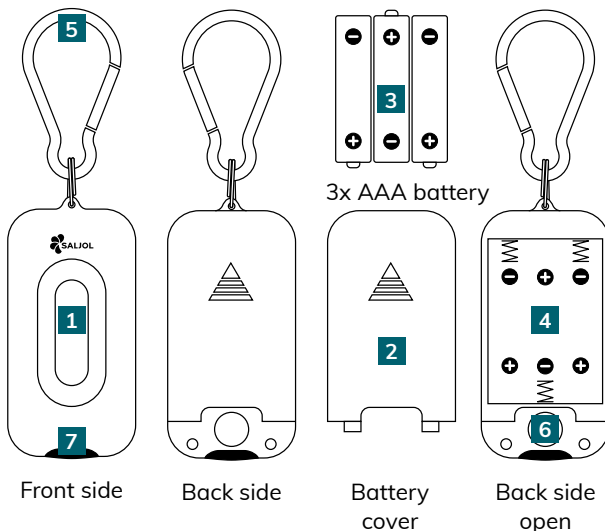
Article number: 4058789013090

Commissioning

Carefully remove the battery cover (2) on the back of the LED light in the direction of the arrow. Insert three AAA batteries (3) into the battery case (4), make sure that the polarity is correct. Close the battery cover (2) again, making sure that it is secure and tight.

The LED light can be attached to the loops of Saljol rollator bags at the front or back by using the snap hook (5).

Use the on/off switch (7) to turn the LED light on and off.

Front side

Back side

Battery cover

Back side open

Technical data

Size	75 x 36 x 15 mm
Battery life	approx. 20 hours
Brightness	120 lm
Weight with batteries	51 g
Weight without batteries	23 g

Information correct at time of printing. Technical specifications are subject to change without notice. Technical drawings and photos may differ from reality. ©2023 Saljol GmbH: all rights reserved. Reprints, including excerpts, are only permitted with the permission of Saljol GmbH.

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Manufacturer: Saljol GmbH